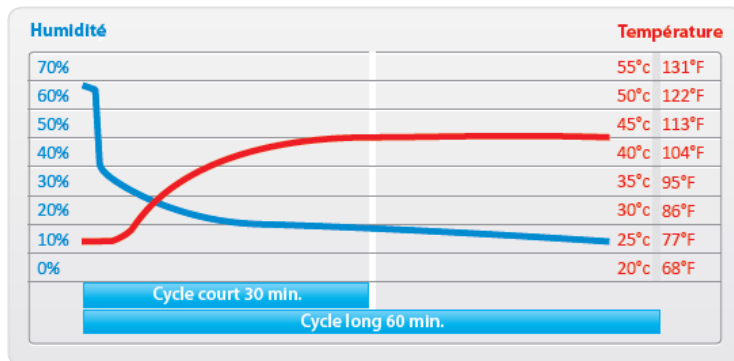
PerfectDry Lux

- Design in 2012
- First commercialization in January 2013


- Description of the PerfectDry Lux:

The device is a drying (by ventilation and heating) and cleaning (by UV-C 360°) device compatible with all hearing aid models. The PerfectDry Lux provides two drying cycles :

- Short cycle of 30 minutes to ensure effective drying under normal / standard conditions.
- Long cycle of 60 minutes, allowing an effective drying in unfavorable conditions (high perspiration, high ambient humidity, ...)

- Dimensions:

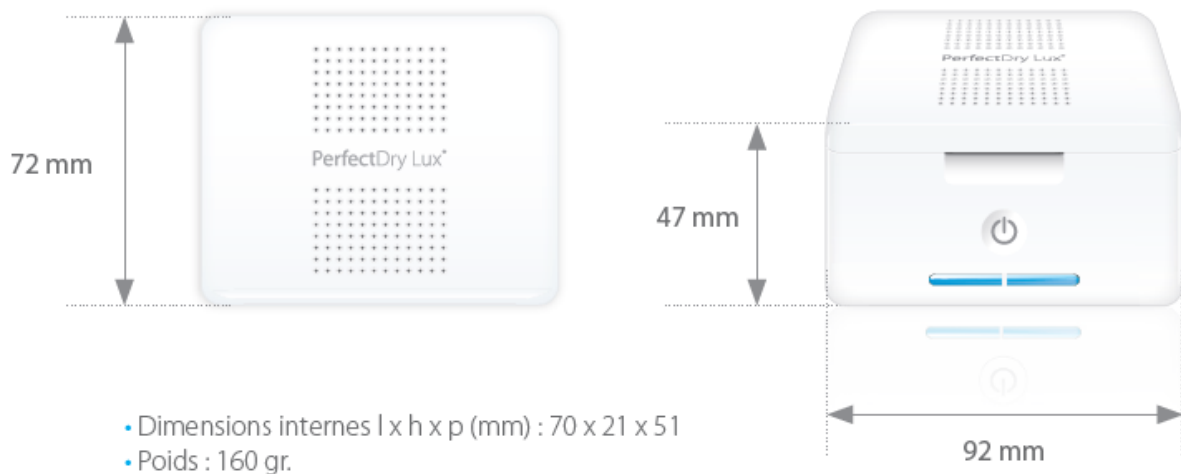- Indications:

PerfectDry Lux is indicated for:

- Dehumidifying hearing aids, in standard or unfavorable conditions, in order to avoid malfunctions due to humidity.
- The elimination of microorganisms on the surface of hearing aids in order to limit the pathologies of the auditory canal related to the wearing of hearing aids.

- Documentations:

- Datasheet: FIT000050 (FR) + FIT000095 (EN)
- Instructions for use MG0 NOTLUX-FR + MG0 NOTLUX-UK

PerfectDry Lux Optimized

- Design in February – March 2015

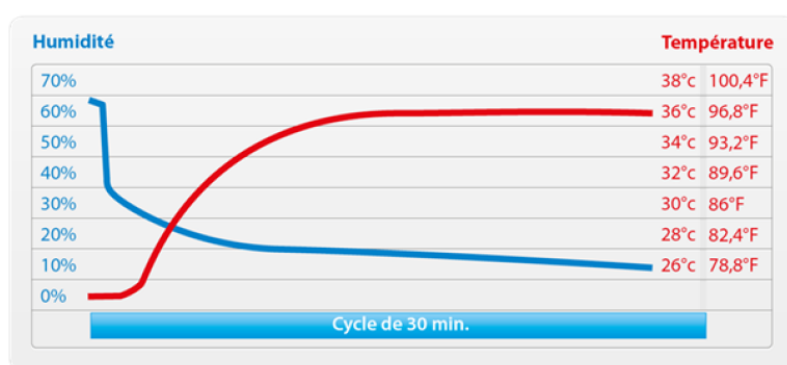
- First commercialization in December 2015

- New features compared to the previous generation:

The new features of the PerfectDry Lux Optimized are the simplified user interface, the proposal of a single cycle and the optimization of drying performance (effective drying in 30 minutes under all conditions).

- Description of the PerfectDry Lux:

The device is a drying (by ventilation and heating) and cleaning (by UV-C 360°) device compatible with all hearing aid models. The PerfectDry Lux Optimized provides optimal drying of the hearing instruments in only 30 minutes, regardless of the initial conditions.

- Dimensions:

- Indications:

The PerfectDry Lux is indicated for:

- Dehumidifying hearing aids to prevent malfunctioning due to humidity.
- The elimination of microorganisms on the surface of hearing aids in order to limit the pathologies of the auditory canal related to the wearing of hearing aids.

- Documentations:

- Datasheet: FIT000270 (FR) + FIT000229 (EN)
- Instructions for use: BAT02215 (FR) + BAT02211 (EN)

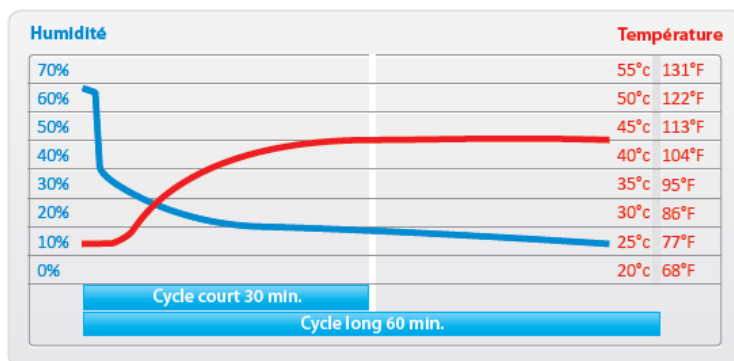
PerfectDry Lux

- Design in 2012
- First commercialization in January 2013


- Description of the PerfectDry Lux:

The device is a drying (by ventilation and heating) and cleaning (by UV-C 360°) device compatible with all hearing aid models. The PerfectDry Lux provides two drying cycles :

- Short cycle of 30 minutes to ensure effective drying under normal / standard conditions.
- Long cycle of 60 minutes, allowing an effective drying in unfavorable conditions (high perspiration, high ambient humidity, ...)

- Dimensions:

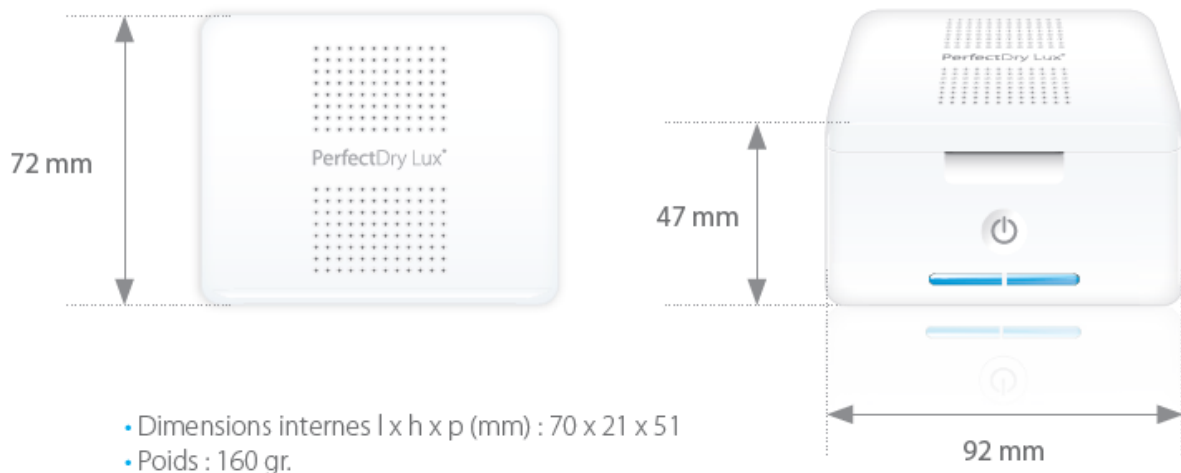- Indications:

PerfectDry Lux is indicated for:

- Dehumidifying hearing aids, in standard or unfavorable conditions, in order to avoid malfunctions due to humidity.
- The elimination of microorganisms on the surface of hearing aids in order to limit the pathologies of the auditory canal related to the wearing of hearing aids.

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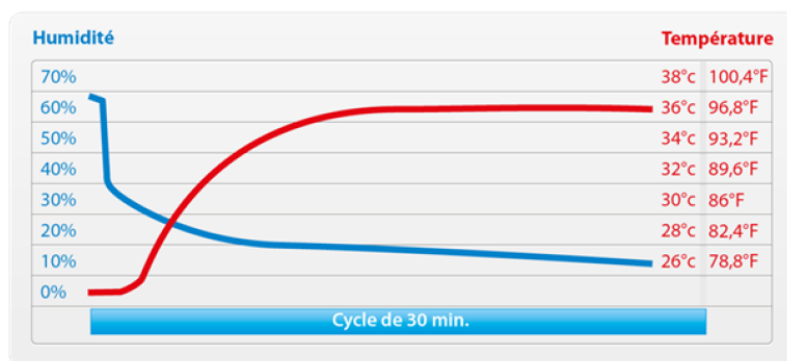
Presentation

PerfectDry Lux® is designed for the maintenance and storage of all types of hearing aids. Its modern and sleek design combined with the best technology makes it exceptionally convenient to use. Its efficiency in drying and disinfecting has been proved by many independent tests.

Specifications

Auto-stop cycle system

- Sensor detecting opening of the lid

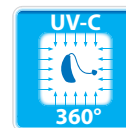
Microprocessor control

- Optimum temperature control
- Disinfection system steering
- Safety management

Forced-air fan drying system

- Temperature self-regulated at 113 °F +/-3 with a chip
- Controlled ventilation
- Optimized and homogeneous air flow
- Extraction of moisture

360° UV-C Lamp

- Shortwaves disinfection (253.7nm)
- Peripheral light 360° (2 Watts)
- Eliminates over 99% of viruses and bacteria (proven certified laboratory)
- Confirmation test on no damages on hearing aids surfaces (nano coating and varnishes)


One touch control

- Touch button with curved sensor
- Intuitive selection of operating cycles

Light indicator

- Blue LEDs indicating status

Evaporation	30 min.	60 min.
Test with water in a tube diameter 2mm.	10 mm ³	20 mm ³
Test with water in a cup	0,2 cm ³	0,4 cm ³

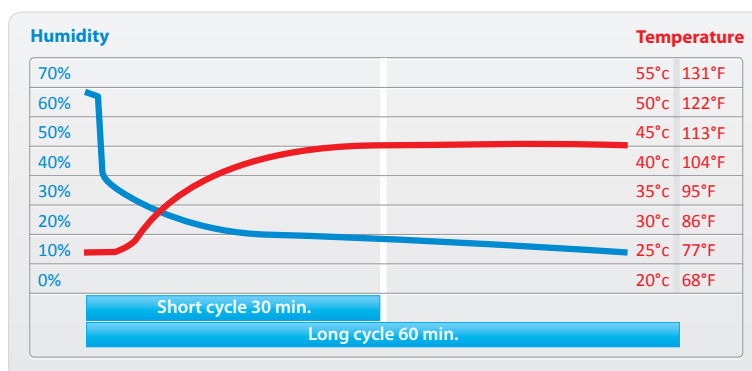Tests with an ambient temperature of 25°C / 77°F and a humidity of 60%

Directives

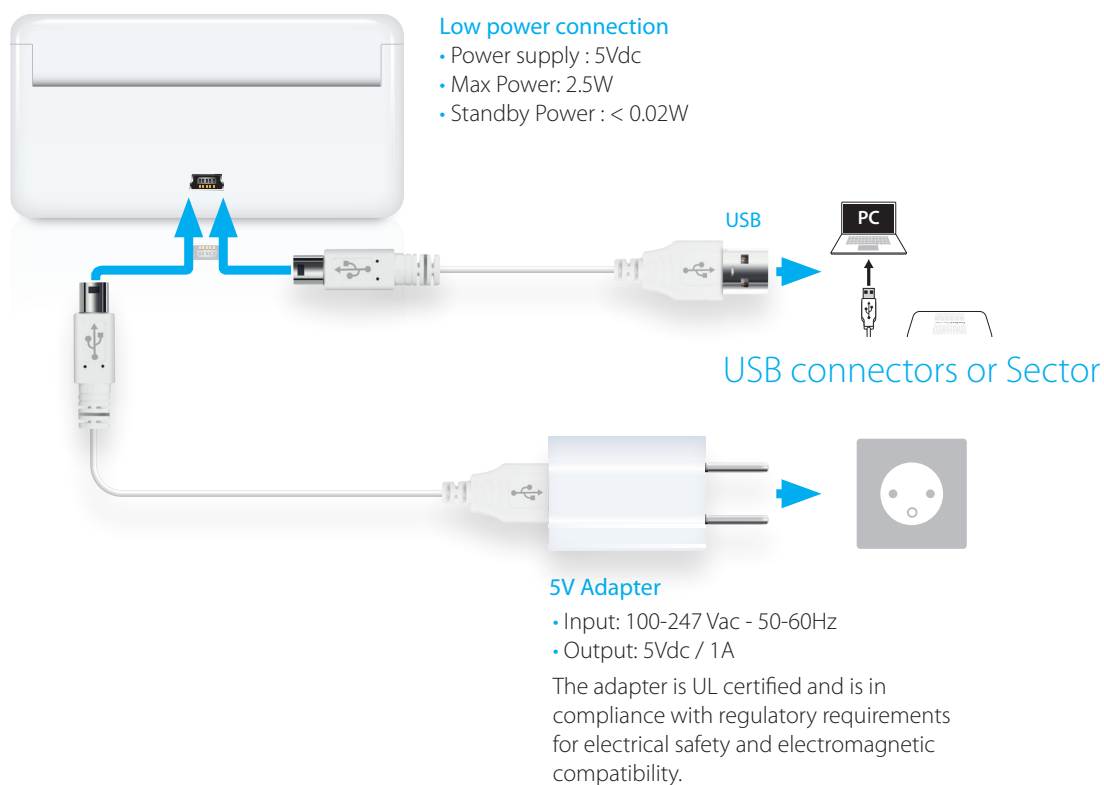
PerfectDry Lux® meets the requirements of the electrical equipment:

- Directive 2014/35/EU - Harmonization of the laws of the Member States relating to the making available on the market of electrical equipment designed for use within certain voltage limits.
- Directive 2014/30/EU - Harmonization of the laws of the Member States relating to electromagnetic compatibility.
- Directive 2011/65/EU - Restriction of the use of certain hazardous substances in electrical and electronic equipment.
- Directive 2012/19/EU - Waste electrical and electronic equipment (WEEE).


Standards

- IEC 61010-1:2010 (June 2010) - Safety requirements for electrical equipment for measurement, control, and laboratory use.
- IEC 61326-1:2012 (July 2012) - Electrical equipment for measurement, control and laboratory use – EMC requirements.

Electrical specifications

Mecanical specifications

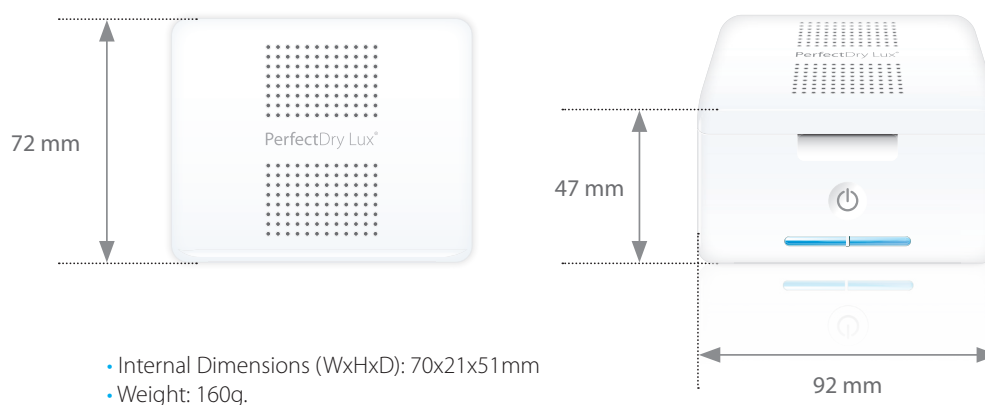Properties

To perform maintenance with high efficiency, PerfectDry Lux® incorporates a microprocessor that controls optimally the different technologies involved :

- **The drying** with hot air. Therefore, the power and the necessary drying times are reduced. Temperature maintenance chamber is controlled by the electronic system in order to obtain a constant temperature of 45°C / 113 °F regardless of the ambient temperature so as not to damage the hearing aids.
- **The disinfection** system Ultra Violet (UV-C) with short wavelengths with the annular arrangement at 360° allows to reach all surfaces of the hearing aid. The exposure to UV-C light allows the elimination of more than 99% of viruses and bacteria and greatly reduces itching and infections of the ear canal. The lamp lights up during the first five minutes of the maintenance cycle and turns off automatically. The efficacy of the disinfection system (bactericidal and fungicidal) is certified by a laboratory 'internationally recognized (Intertek). The tests are consistent with the requirements for disinfection of medical field. The same laboratory performed tests to ensure that the disinfection system does not damage hearing aids.

Indications

The PerfectDry Lux® is indicated to dry all types of hearing systems and earmolds to avoid failure due to moisture .

Regular use prevents skin irritations and eczema and increases the lifetime of the hearing system and preserve their effectiveness.

Usage Tips

Clean your hearing aids to remove excess dirt. If possible open the battery compartment of your hearing aid to enable full dehumidification then place in the PerfectDry Lux®.

Close the lid of the PerfectDry Lux®.

Then tap the operation button touch to select the desired quick (30min) or intensive (1h) cycle. The two blue lights indicate the current cycle.

The PerfectDry Lux® automatically stops at the end of the maintenance cycle (lights switch off). It is possible to stop the maintenance cycle by opening the lid.

Precautions

Put PerfectDry Lux® on a dry, stable support.

To unplug, grasp the plug, not the cord.

Clean with a soft cloth. Do not use abrasives to clean PerfectDry Lux®. Do not spray liquid into the unit. Keep PerfectDry Lux® closed when not in use to prevent dirt deposits.

PerfectDry Lux® should be used by a person in full possession of his intellectual capacities.

Keep out of reach of children.

Any intervention other than cleaning and normal maintenance by the customer, must be done by the aftermarket manufacturer or by an authorized professional service.

Do not put PerfectDry Lux®, the power cord or plug in water or any other liquid. If the power cord or plug are damaged, do not use the device and change immediately.

Replace your AC adapter with a genuine model. Never use detergents or corrosive to maintain your PerfectDry Lux®.

Warranty

Built to last, each PerfectDry Lux® follows a multi-level test process to ensure the specified operation.

PerfectDry Lux is guaranteed for a period of 36 months from the date of purchase.

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- Entrée : 100-247 Vac – 50-60Hz - 0,15 A
- Sortie : 5Vdc / 1A

Presentation

The **PerfectDry Lux** is a Medical Device for the maintenance and storage of all kind of hearing aids. Its modern and sleek design combined with the best technology makes it exceptionally convenient to use. Its efficiency in dehumidifying in only 30 minutes has been proved by many independent tests.

Specifications

Auto-stop cycle system

- Sensor detecting opening of the lid.

Microprocessor control


- Dehumidification system.
- UV-C system.
- Automatic shutdown at the end of the cycle.

Patented pulsed hot air dehumidification system

- The temperature is self-regulating in order to not exceed 37°C/99°F thanks to the microprocessor.
- Controlled ventilation.
- Optimized and homogeneous air flow.
- Extraction of moisture.

Evaporation	30 min.
Test with water in a tube diameter 2mm.	10 mm ³
Test with water in a cup	0,2 cm ³

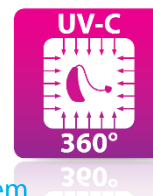
UV-C system

- Shortwaves (253.7nm).
- Peripheral light 360° (2 Watts).
- Confirmation test on no damages on hearing aids surfaces (nano coating and varnishes).
- Automatic shutdown when the cover is opened.

One touch control

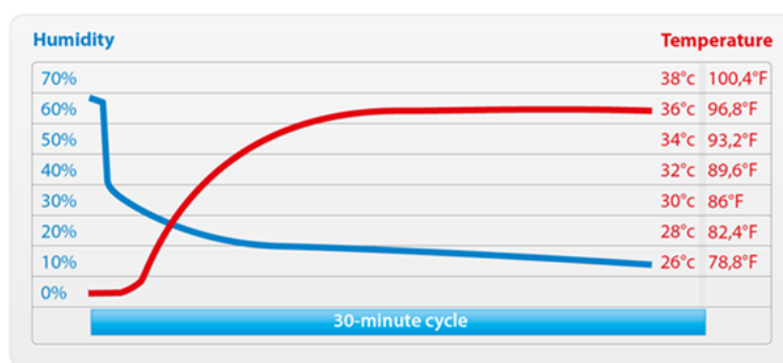- Touch button with curved sensor.

Test with an ambient temperature of 77°F and a humidity of 60%.

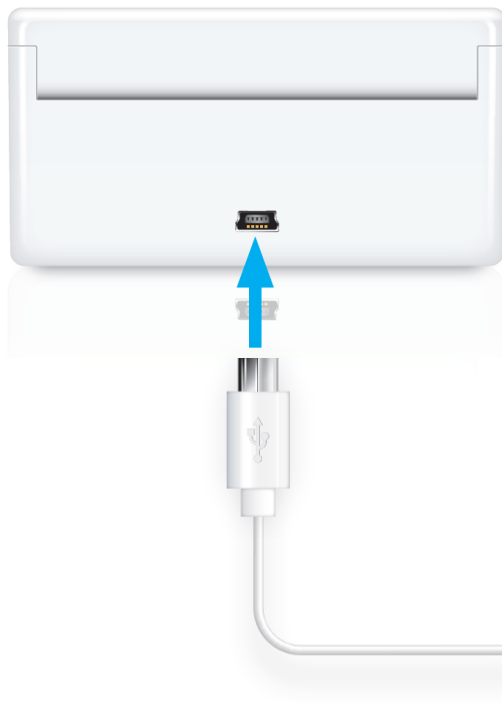
Directives

The PerfectDry Lux meets the requirements of:

- Directive 93/42/EEC – Concerning Medical Devices, reviewed and amended by Directive 2007/47/CE.
- Directive 2014/35/UE – Harmonization of the laws of the Members States relating to the making available on the market of electrical equipment designed for use within certain voltage limits.

- Directive 2014/30/EU – Harmonization of the laws of the Member States relating to electromagnetic compatibility.
- Directive 2011/65/UE – Restriction of the use of certain hazardous substances in electrical and electronic equipment.
- Directive 2012/19/EU – Waste electrical and electronic equipment (WEEE).

Electrical specifications

Low power connection

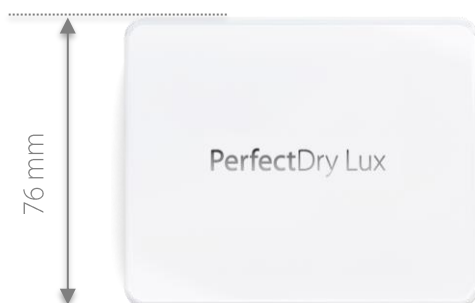
- Power supply: 5Vdc
- Max power: 4W
- Standby Power: <0.02W

5V adapter

- Input: 100-240 Vac – 50-60Hz
- Output: 5Vdc
- The adapter is certified and is in compliance with regulatory requirements for electrical safety and electromagnetic compatibility.

Mechanical specifications

- Internal Dimensions (W x H x D): 70 x 21 x 51 mm
- Weight: 160g

Properties

PerfectDry Lux is a Medical Device that guarantees effective hearing aid care. To this extend, it incorporates a microprocessor to optimally control the different technologies involved:

- **The dehumidification** with hot air. Therefore, the power and the necessary dehumidification times are reduced. Temperature maintenance chamber is controlled by the electronic system which allows to not exceed 37°C/99°F regardless of the ambient temperature so as not to damage the hearing aids.
- **The UV-C system** with short wavelengths and the annular arrangement at 360° allows to reach all surfaces of the hearing aid. The exposure to UV-C light allows the elimination of more than 99% of fungi and bacteria and greatly reduces itching and infections of the ear canal. The lamp lights up during the five minutes of the maintenance cycle and turns off automatically. The efficacy of the UV-C system is certified by a laboratory internationally recognized (Intertek). The tests are consistent with the requirements for UV-C of medical field. The same laboratory performed tests to ensure that the UV-C system does not damage hearing aids.

Indications

The **PerfectDry Lux** is indicated to dehumidify all types of hearing systems and earmolds to avoid failure due to moisture.

Regular use increases the lifetime of the hearing system and preserve their effectiveness.

Usage Tips

Clean your hearing aids to remove excess dirt. If possible open the battery compartment of your hearing aid to enable full dehumidification, then place in the **PerfectDry Lux**.

Close the lid of the **PerfectDry Lux**. Then tap the operation button touch to launch the cycle (30 minutes). The **PerfectDry Lux** automatically stops at the end of the maintenance cycle (light switches off). It is possible to stop the maintenance cycle by opening the lid.

Precautions for use

Put **PerfectDry Lux** on a dry, stable support.
To unplug, grasp the plug, not the cord.

Clean with a soft cloth. Do not use abrasives to clean **PerfectDry Lux**. Do not spray liquid into the unit. Keep **PerfectDry Lux** closed when not in use to prevent dirt deposits.

Keep out of reach of children. Any intervention other than cleaning and normal maintenance by the customer, must be done by the aftermarket manufacturer or by an authorized professional service.

Do not put **PerfectDry Lux**, the power cord or plug in water or any other liquid. If the power cord or plug are damaged, do not use the device and change immediately.

Replace your AC adapter with a genuine model.

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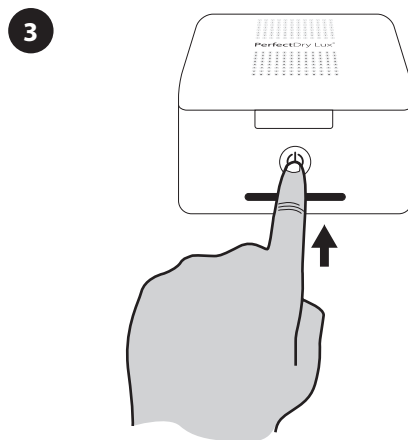
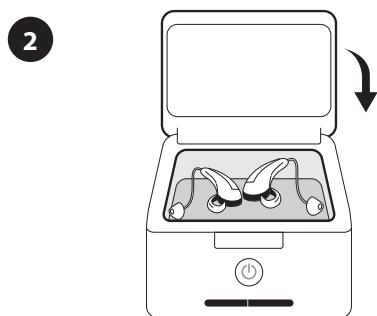
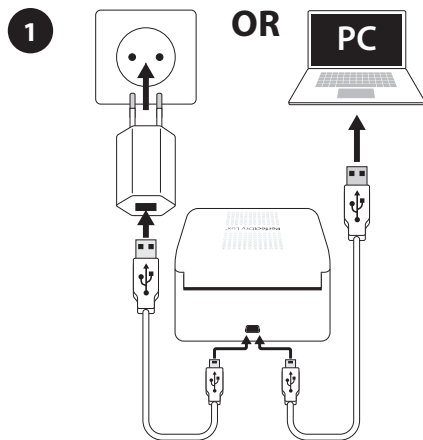
➔ **Touch x1 = Fast cycle**
30 min.

➔ **Touch x2 = Intensive cycle**
1h

1 Plug the USB power supply on the back of the PerfectDry Lux®. Then plug the cable on the USB slot or on the electrical socket. The pilot lights blink, indicating that the device is on.

2 Before placing your hearing aids into the PerfectDry Lux®, clean and wipe them to remove earwax and impurities. If it is possible, place your hearing aids with the battery housing open for an optimum dehumidification process. Close the lid of the PerfectDry Lux®.

3 Touch the start button to select your drying cycle:

➔ **Touch once** to start a fast drying cycle of 30 minutes, the light on the left turns on.

➔ **Touch twice** to start an intensive drying cycle of 1h, both lights turn on.

After one minute, without any action, the PerfectDry Lux® will activate the stand-by mode and the lights will turn themselves off. Touch the start button once to reactivate the device, then select the drying cycle you need.

The PerfectDry Lux® will stop automatically at the end of the drying cycle and the pilot lights will turn themselves off. If you need to stop an ongoing drying cycle, you simply need to open the lid and it will stop automatically (security feature).

Precaution of use

- Place the PerfectDry Lux® on a stable and dry base.
- Do not stare directly at the UV-C lamp located inside the drying chamber.
- To unplug the device from the socket, pull on the power plug directly but do not pull on the power cord.

Maintenance advice

Clean with a soft cloth. Do not use any powerful or abrasive cleaning products to clean the PerfectDry Lux®. Do not spray any liquid into the box. Keep the PerfectDry Lux® with the lid closed when you are not using it to avoid the deposit of impurities on the UV-C lamps.

MG0 NOTLUX-UK_160203

PerfectDry Lux®

Security Instructions

Read the instructions very carefully before the first use and keep it for next uses.

Any use not in accordance with the instructions will free the manufacturer from any responsibility.

The PerfectDry Lux® must be used by a person with full intellectual capacities. No other restriction applies to the user.

Keep out of reach of children.

Any intervention, other than usual cleaning and maintenance from the user, must be operated by the Aftersales Service or a professional.

Never put your PerfectDry Lux®, power supply or cable under water or any other liquid. If the power plug or the cable are damaged, do not use the device. Replace your power supply with an original one. Never use detergent or corrosive product to clean your PerfectDry Lux®.

Warning : Any use not in accordance with these instructions will cancel the warranty.

The manufacturer cannot be held responsible for damages caused by a misuse or bad manipulation of the product.

Description

The PerfectDry Lux® is an electronic device for the maintenance and the storage of all kind of hearing aids.

This maintenance is ensured by the combination of a forced-air fan drying system and UV-C rays disinfection.

Note:

- The UV-C lamp is a surface disinfectant and it has few or no penetrating qualities, which means it cannot damage the earmold or plastics of the hearing aids.

UV-C disinfection

The PerfectDry Lux® is equipped with a short emitting waves UV-C lamp, whose 360° circular disposition enables to reach all the surfaces of the hearing aids.

The exposition of surfaces to the UV-C light allows to destroy 99% of fungi and bacteria, and dramatically reduces itchiness and infections of the ear canal.

The lamp will turn itself on during the first five minutes of the drying cycle, then turn itself off automatically.

Dehumidification - Drying

The PerfectDry Lux® is equipped with a forced-air fan drying system, which enables to reduce the power and time necessary for the drying cycle.

The temperature of the drying chamber is controlled by an electronic system in order to maintain a constant temperature of 45°C, whatever the ambient temperature surrounding the device, and to prevent the hearing aids from any damages.

Technical data

Features

- External dimensions l x h x p (mm): 92 x 48 x 75
- Internal dimensions l x h x p (mm): 70 x 21 x 51
- Weight (g): 160
- Power supply: 5Vdc / 0,5A
- Maximum power: 2.5W
- Stand by power: <0.02W
- UV-C wavelength: 253.7nm
- Acoustic Level <20 dBA
- Drying temperature: 45°C +/-3
- Operating temperatures: from 12° to 40°C
- Storage conditions: from -20° to 65°C
- Software Version 01.00

Power connector

- Input: 100-247Vac - 50-60Hz - 0.15A
- Output: 5Vdc / 1A

Identifying information

You will find the data sheet under your PerfectDry Lux®.

- V** Abbreviation signification:
Volts (Power supply tension)
- A** Ampère (Energy consumption)

 USB power symbol

 Inside use

 This product is made of recycled ABS- code 7

 This product complies with the applicable European directives.

 **ELIMINATION**
Throw the old material out in an ecological way.
The old devices are made with recycling material which must be recycled. Be careful to throw your old devices out in the right recycling system.

User Instructions

PerfectDry Lux®

UV-C disinfection
and quick dehumidification
of hearing aids

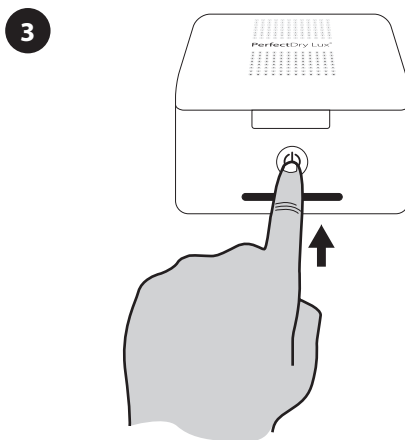
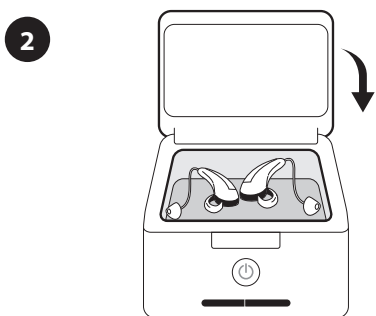
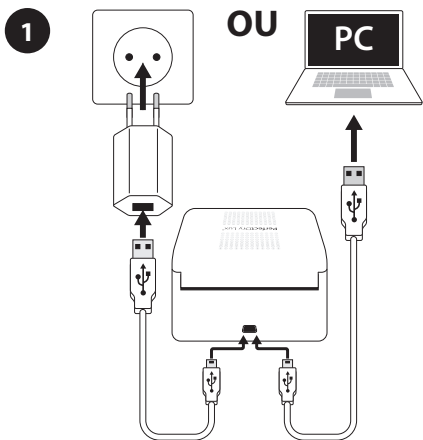
This box set includes:

- 1 PerfectDry Lux®
- 1 USB cable
- 1 Power connector
- 1 User manual

Guarantee

PerfectDry Lux® is guaranteed for 36 months, except for the power supply (24 months), from the date of purchase. For any failure or breakdown, contact your hearing aid specialist. A purchase proof will be asked from Aftersales Service during the guarantee period. This guarantee will not be valid for any use not in accordance with these

instructions, for an accident, a liquid introduction, a negligence, a maintenance lack or any other circumstance caused by the user. Furthermore the guarantee will not be valid for any problem caused by lightnings or a variation in the electrical voltage. As a precaution we recommend to unplug the device in case of storms.

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
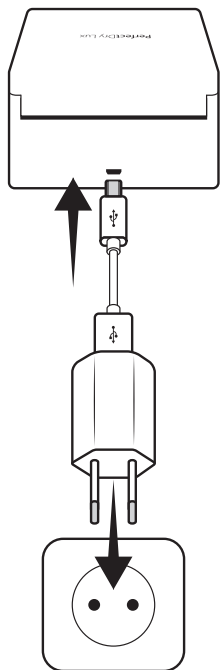
1

Connect the PerfectDry Lux. The blue indicator lights to show that it is powered on.
If no actions occur for 30 seconds, the indicator turns off.


2

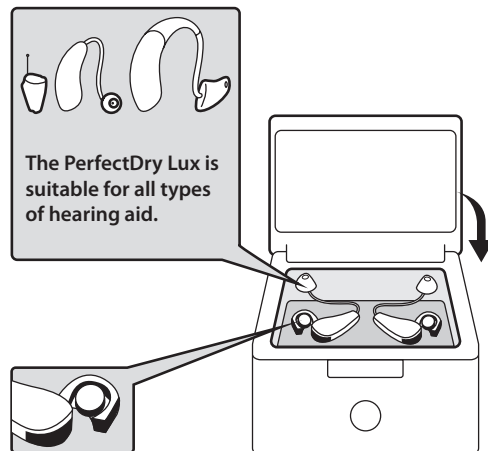Clean your hearing aids using a specific cleansing wipe to eliminate excess impurities. Then open the battery compartments on your hearing aids (the batteries can be left inside the hearing aids during the cycle). Put your hearing aids then close the cover.


3

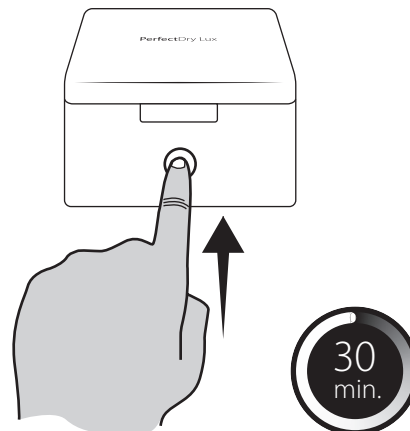Press the touch button once to start the cycle. The blue indicator flashes slowly, indicating that the box is running. The cycle will only run if the cover is closed. The PerfectDry Lux stops automatically at the end of the maintenance cycle (30 minutes). The indicator turns off.

Make sure the cover is closed when you start the cycle.
Also make sure you do not open the box when it is running, if you do so the PerfectDry Lux will stop in order to maintain safety.

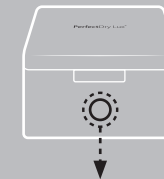Meaning of the light indicators

BLUE Flashing >

Running

BLUE On >

In standby mode

Identification data

This information can be found on the lower part of your PerfectDry Lux.

Meaning of abbreviations:

V Volts (Power supply voltage)
A Ampere (Power consumption)

 USB power supply symbol

 Indoor use

 The product is made from ABS with a recycling code of 7

 This product is compliant with all European Directives applicable to Medical Devices

 **DISPOSAL**
Dispose of previous appliances ecologically.

Old appliances contain recyclable materials that can be recovered. Therefore, please dispose of your old appliances using the appropriate waste disposal service.

PerfectDry Lux

Function details

360° UV-C

The PerfectDry Lux has a short-wave UV-C lamp of which the 360° circular layout makes it possible to reach all the hearing aid surfaces.

Hearing aid care is obtained by a combination of fan-pulsed hot air dehumidification and 360° UV-C.

The lamp lights up for the first five minutes of the maintenance cycle, then turns off automatically.

Dehumidifying

The PerfectDry Lux has a patented pulsed air dehumidification system. This means that the required power and drying time is reduced. The care chamber temperature is controlled by an electronic system in order to obtain a maximum temperature of 37°C whatever the room temperature, thereby preventing damage to the hearing aids being cleaned.

In order to maintain optimum hearing aid quality, the daily use of the PerfectDry Lux is strongly recommended.

Medical Device Directive

This box meets the requirements of directive 93/42/EEC on Medical Devices.

Technical data

Functions

Internal (lxwxd): 70x21x51 mm

External (lxwxd): 93x48x76 mm

Weight (g): 144g

UV lamp: 253.7nm

Max. power: 4W

Standby power: 0.04W

Software: 00 PDL-LOG-02.00

Storage conditions from -20°C to +65°C

Number of cycles: 1

Drying temperature: 37° C

Cycle duration: 30 min.

Ambient operating temperature:

From 12°C to 40°C

Power Supply:

Input: 100-240Vac 50-60Hz / Output: 5Vdc

Connectors: USB cable - micro USB

Safety instructions

Keep out of the reach of children. Any work, other than cleaning and routine care by the client, must be carried out by the manufacturer's customer service or by an approved professional. Do not place the PerfectDry Lux, the power supply cable or the plug in water or any other liquid. If the power supply cable or the plug are damaged, do not use the device and replace them immediately. Replace your mains adapter with an original model.

Place the PerfectDry Lux on a flat, dry and stable surface. Do not directly look at the UV-C lamp located in the dehumidification chamber. When disconnecting, pull the plug and not the cable.

Never use detergent or corrosive products to clean your PerfectDry Lux. Failure to follow these instructions will void the guarantee. The manufacturer cannot be held liable for damage caused by incorrect use or handling.

Product care advice

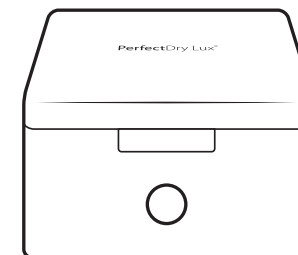
Clean using a soft cloth. Do not spray liquids into the device. Keep the PerfectDry Lux closed when not in use to avoid impurities from being deposited.

Guarantee

36 months - (cable and transformer included). In the event of a malfunction, contact your hearing-aid specialist. Proof of purchase will be requested for all work by the customer service during the guarantee period. This guarantee does not apply to defects caused by an accident or incident or similar damage, by introducing a liquid, negligence, unsuitable use, dismantling of any kind, lack of care or any other circumstance caused by the user. Furthermore, this guarantee will not apply to damage caused by lightning or any other electric voltage variation. As a precaution, please disconnect the PerfectDry Lux in the event of a thunderstorm.

User instructions

PerfectDry Lux

Fast dehumidification
of hearing aids,
with 360° UV-C

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